

ENGLISH

Read all of the instructions for use carefully before you start using this medical device.

- If you have any further questions, please contact your doctor, therapist, pharmacist or the manufacturer FROXIMUN AG directly.
- This medical device is available over the counter without a prescription.

1. What is SKIN POWDER NC 100 and what is it used for?

SKIN POWDER NC 100 for external use is a natural medical device to support wound healing in cases of superficial skin injuries or skin irritations.

Areas of application:

- Adsorption of wound exudate
- Weeping rashes
- Minor, superficial skin wounds or minor abrasions (do not use on open wounds)

2. What you need to know before using SKIN POWDER NC 100?

- Do not open the container until immediately before use.
- Make sure that the container is always closed.
- The skin powder may only be used for the purpose specified in these instructions for use.
- Do not sprinkle powder in the eyes, ears or nose.
- Do not inhale powder.

Do not use SKIN POWDER NC 100:

In case of a known allergy or hypersensitivity to the ingredient. (see point 6: Ingredients)

3. How to use SKIN POWDER NC 100?

- Sprinkle powder onto the previously cleansed area of the body as required.
- Repeated daily application is recommended.
- Do not repeat the application until you have cleaned the wound.
- Clean the treated area after 24 hours at the latest.

Duration of use

The application can be continued until the wound has healed. Please consult a doctor or healthcare professional if your symptoms do not improve.

Countermeasures in the event of incorrect use, overdose and side effects:

- There are no known overdoses.
- In the event of eye contact, rinse with clean water; consult a healthcare professional if necessary.

4. Possible side effects

There are no known side effects.

Reporting of side effects

If you get any side effects, talk to your doctor, healthcare professional or pharmacist. Please also report this to FROXIMUN AG using the contact details below:

Manufacturer and reporting centre: FROXIMUN AG, Neue Straße 2a,
D - 38838 Schlanstedt

Phone: +49 39401 632-0

Fax: +49 39401 632-199

E-mail: anwender@froximun.de

Person responsible for regulatory compliance (PRRC): +49 39401 632-320

5. How to store SKIN POWDER NC 100?

- Keep out of reach of children
- Store in a dry place between 5°C - 40°C
- Protect from sunlight
- Use by: see embossing/printing on the edge of the label and on the folding carton

6. Further information

Contents and ingredients

- Package contents: 2 containers with 4g of powder; instructions for use printed on the inside of the label
- Ingredients: MANC® (natural zeolite clinoptilolite)

Information on MANC®:

- 100% natural, gluten-free, lactose-free
- MANC® is a modified aluminosilicate.
- kosher, halal: We hereby confirm that MANC® is manufactured in a process from mineral raw materials. The product does not contain animal ingredients, blood or its by-products, or alcohol.

7. Manufacturer information

FROXIMUN® AG, Neue Straße 2a, 38838 Schlanstedt, Germany

Phone: +49 39401 632-0, Fax: +49 39401 632-199

E-mail: info@froximun.de

These instructions for use were last revised on: 2026-07-24

ENGLISH


Warnings and precautions



Manufacturer



Observe the instructions for use

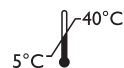

Store between
5°C - 40°C



LOT,
batch number



Store in a dry place



Keep out of reach
of children



Use by



Protect from sunlight



Medical device